



Flexabed Hi-Low SL

The ideal bed for those in home healthcare, or assisted living environments, the Hi-Low SL can be raised and lowered vertically with a touch of a button — making the Hi-Low SL model ideal for customers who require additional assistance when entering or exiting the bed and for adjusting the bed to a comfortable working height for care-takers to provide proper care for the bed occupant.

The Hi-Low SL also allows couples who require different features in their beds to still be able to sleep in the same bed. Instead of one spouse being forced to sleep separately in a hospital bed, a Flexabed Hi-Low SL can be paired with a Premier or Value-Flex bed allowing spouses to share the same top sheets and comforter!

The Flexabed Hi-Low SL offers many of the same features as a hospital bed but also offers the luxury and comfort of an adjustable bed. If you need relief from the symptoms of injury, chronic pain or illness, or are coming home after a surgery, there's nothing more refreshing than an adjustable bed. You can raise or lower your head and feet to find your favorite position. The Flexabed Hi-Low SL is the best way to provide you or your loved one the most comfortable bedding solution for home use that a traditional flat bed or hospital bed cannot provide.

Every Hi-Low SL model comes with

- a limited lifetime warranty.
- your choice of one-touch wireless or wired remote.
- whisper-quiet motors with optional dual timed massage and optional under-bed lighting.
- a 6 inch foundation profile, high-density, heavy duty padded base made with Advantech® material that will not warp, break or squeak.
- a Premier mattress featuring a beautiful knit fabric quilt panel with a tack and jump pattern and a stylish knit border with designer taped edges. It is thicker and more luxurious than ever before. Choose from several different mattress constructions.
- optional Bluetooth connectivity which allows compatible smartphones the ability to control the bed.
- a cover for the mattress retainer bar at the foot of the bed.
- the ability to be paired with a Flexabed Premier or Value-Flex — perfect for a spouse.
- factory-installed mounting plates for easy installation of side rails at the head and the foot of the bed.
- locking casters or leg pads.
- optional side rails with protective covers.

The Flexabed Hi-Low SL is available in several different standard sizes and lengths. Custom sizes are also available*. Mattresses are also available in several different constructions that are tailored to fit your individual needs . See page 9 for a complete listing of specific bed dimensions and heights. (*Additional charges apply on custom orders and custom sizes are non-returnable.)

A Flexabed adjustable bed should not be considered an alternative to a hospital bed. A physician should be consulted when health issues are involved as to the appropriateness of an adjustable bed.

EASE OF USE

How This Model Works



1. Ease onto the bed with both feet on the floor.

The Flexabed Hi-Low SL can be lowered to a height that allows easy transfer in and out of the bed. Optional side rails can be added that also serve as a lift assist during transfer. Fully raise the head section of the bed with your remote during transfer which allows the occupant to swing their legs in or out of the bed without having to support the weight of their torso.


2. Raise the bed vertically with the touch of a button.

Being able to raise or lower the bed vertically allows the user to adjust the bed to a height that makes transfer in or out of the bed easiest on them. For users who have a caretaker, a Hi-Low SL bed allows the caretaker to adjust the bed vertically to a comfortable working height to assist the occupant with feeding, changing, adjustment, etc.

3. Find the most comfortable position for you.

Raise or lower your head and feet with our whisper quiet dual motor technology and activate the optional dual timed massage featuring wave and pulse technology at your back and legs. Relax and enjoy.

Hi-Low Bed Heights

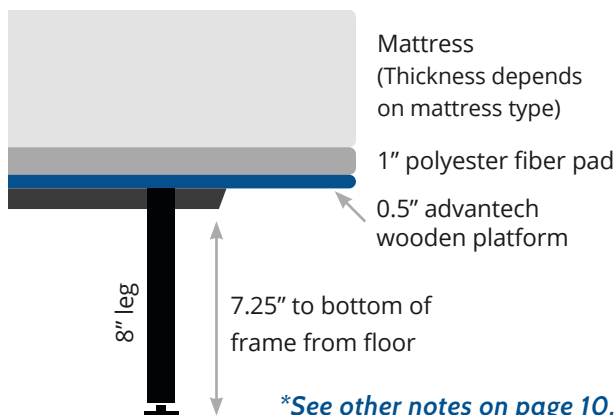

Height Fully Lowered

	With Leg Pads	With Casters
Floor To Top of Foundation (no mattress)	11"	13.25"
Floor To Top of Low Profile Mattress	16.5"	18.75"
Floor To Top of Quilted Top Mattress	20-5/8"	22-7/8"
Floor To Top of Gel Mattress	21"	23.25"

Height Fully Raised

	With Leg Pads	With Casters
Floor To Top of Foundation (no mattress)	18.5"	20.75"
Floor To Top of Low Profile Mattress	24"	26.25"
Floor To Top of Quilted Top Mattress	28-1/8"	30-3/8"
Floor To Top of Gel Mattress	28.5"	30.75"